29 cm

2 cm

5,5 cm

2 cm

10 cm

2 cm

5,5 cm

2 cm

Manufacturer: 
Medoffice Saglik Endustri Anonim Sirketi
Yazibasi Mah. Kuşçu Cad. No: 9/1-1 Torbalı / İzmir - TÜRKİYE
Tel: +90 232 431 31 12 Fax: +90 232 431 31 09
Mail: info@medoffice.com.tr
Web: www.medoffice.com.tr

Chemical Ingredients:
Aqua, Dimethicone(%3), Linoleamidopropyl Pg-D Chloride Phosphate (Phospholipid), Lauryl Glucoside, D-Panthenol (Provitamine B5), Allantoin, Aloe Berbadensis Extract, Polysorbate20, Tetrasodium EDTA, Citric Acid, Tocopheryl Acetate (Vitamine E), Chlorhexidine Digluconate, Glycerin, Chamomile Recutita Flower Extract, Potassium Sorbate, Sodium Benzoate

8 683148 052333

Ref: MCP0025

MD

LOT

KEEP DRY

KEEP AWAY FROM SUNLIGHT

DO NOT REUSE

Turkey
Discover the potential
MADE IN TÜRKİYE
صناعة تركية

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pH: 5,5

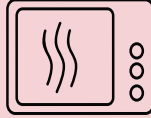
CE ISO 9001 ISO 13485 ISO 22716 ISO 14001

Instructions for Use:
Perineal Area Cleansing Wipes provide cleansing, care and safe effect for the bedridden and intensive care unit patients perineal area. Please wear rubber gloves to avoid contact with the body fluids before use. Open the package from "open here" part on the label. Apply gently because of the sensitivity of the perineal region. Please use a separate wipes for each body area. No need extra water, rinse and other cleaning products. The material used is medical waste. Dispose according to Medical Devices regulations.

Cautions and First Aid :
• Do not use damaged packaging products
• Product is cleaning and care products
• Do not use this product on wounds as a buffer
• Do not reuse the used products.
• Do not use this products for babies from age 0-6 months .
• Breastfeeding or pregnant women must be used under doctors supervision
• Do not use any cleaning material after and before using this product
• Avoid to contact with eyes. If still contacted, wash the eyes with plenty of water

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Optional

MICROWAVE USE IF NEEDED.
Microwave Instructions for Use
1- Put the package in to Microwave.
2- Hold 20 seconds at 700W.
3- Check the temperature of wipes .
4- Do not use higher 35-40 °C

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